




# Safe and stress-free tube feeding for calves

Designed by a vet, the Trusti and Flexi Tubers significantly reduce calf stress and discomfort.

The soft flexible tube is swallowed easily and calf stress is minimised, making the Trusti Tuber very easy to use.

A specially designed safety tip combined with flexible tube enables the calf to swallow easily and the tube to gently pass into the oesophagus, safely passing the airway.

Bruising and scratching of tissue is prevented due to the protective mouthpiece, flexible tube and safety tip design.

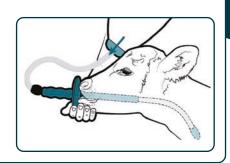





The original Trusti Tuber. Transparency of flexible tube makes it easy to monitor milk flow, and to clean.



ATT01A
Trusti Tuber Starter Kit
(Generation 2)





The Trusti Tuber plus a strap that can be placed behind the calf's ears, securing the mouthpiece for the hand-free operation.



ATT01A
Flexi Tuber Starter Kit
(Generation 2)





A smaller diameter tube for premature or very small calves (<25kg). Purple in colour to differentiate from the standard size.



ATT02A
Trusti Tuber XS Starter Kit
(Generation 2)





The mouthpiece sits comfortably against the muzzle.



Pass the flexible tube through the mouthpiece so tube enters the oesophagus.



Consider the calf size markers a minimum distance to pass the tube through.



The stopper should be positioned to the right hand side of the appropriate mark.



# **During Flow:**

Hold the mouthpiece in place throughout feeding - hold with the calf so if the calf moves the mouthpiece moves with the calf. Do not let the tube pull out more than one inch during fluid flow. Never let the calf drop its head below the level of the stomach.

### Removing the Tube:

Wait at least one second after all liquid has finished running through, then gently remove.

## **IMPORTANT**

Never let fluid flow unless the tube is safely in the oesophagus.



