

# Stress Free

NEW



# Calf Tube Feeding

from

## TRUSTI TUBER STARTER PACK (ANT01A)

Dairy Spares


### UNIQUE DESIGN AND EASY TO USE

Designed by a vet, the Trusti Tuber significantly reduces calf stress and discomfort - soft, flexible tube, swallows easily and the calf struggling is minimised making the Trusti Tuber very easy to use.

- Flexi Tube can be held in place with one hand
- Large 4 litre capacity bottle.
- Transparent flexible tube makes it easy to clean and monitor flow.
- Ergonomic bottle handle - perfect for large and small hands.
- Reliable valve - fast flow even with thick colostrum.
- Shown in trials to almost halve procedure feeding time.



Stopper and calf size markers provide easy guide for tube placement and positioning

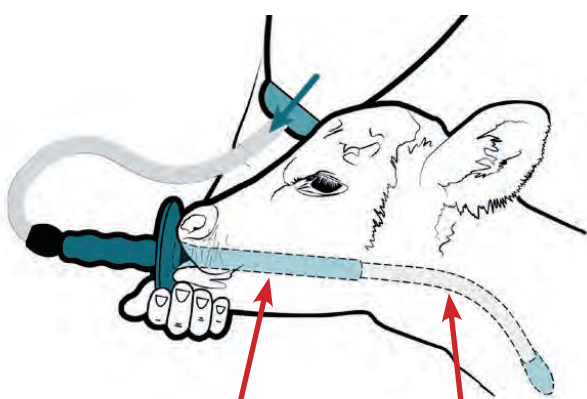

Safety tip

The mouthpiece sits comfortably against the muzzle.



Flexible tube is advanced through mouthpiece so the tube is easily positioned within the oesophagus.



Clip on bottle keeps mouthpiece clean and away from surfaces

# HOW TO USE



# TRUSTI TUBER



The mouthpiece sits comfortably against the muzzle.



Pass the flexible tube through the mouthpiece so tube enters the oesophagus.

Oesophagus.



Stopper and calf size guides for easy positioning.

When the tube has passed into the oesophagus it is safe to raise the bottle.

### IMPORTANT

Never let fluid flow unless the tube is safely in the oesophagus.

#### During Flow:

Hold the mouthpiece in place throughout feeding - hold with the calf so if the calf moves the mouthpiece moves with the calf. Do not let the tube pull out more than one inch during fluid flow. Never let the calf drop its head below the level of the stomach.

#### Removing the Tube:

Wait at least one second after all liquid has finished running through, then gently remove.



### STOPPER POSITION AND CALF SIZE MARKERS:

- Consider the calf size markers a **MINIMUM** distance to pass the tube through.
- The stopper should be positioned to the **RIGHT HAND SIDE** of the appropriate mark.



Do not underestimate calf size as it is better to pass the tube farther than not far enough. If feeding a group of calves we recommend you set the stopper position to the largest calf in the group.



Finger hook on stopper to maintain tube position during feeding.

### SPARES:



Tel: +44 (0) 1948 667676  
 Fax: +44 (0) 1948 666505  
 Email: [sales@dairyspares.com](mailto:sales@dairyspares.com)  
 Website: [www.dairyspares.com](http://www.dairyspares.com)

