

# Stress + Struggle Free Calf Tube Feeding

from

Dairy  
Spares

## FLEXI TUBER 2 IN 1 STARTER KIT

(ANT20A)



The Flexi Tuber provides two methods of calf feeding:

- Place the strap behind the calf's ears to secure the mouthpiece for hands free operation.

OR

- Hold the mouthpiece in place with one hand like the Trusti Tuber.


Use the strap to secure the mouthpiece.



Kinder  
Easier  
Safer  
Faster



Place the mouthpiece so the soft crossbar sits in the corners of the calf's mouth.

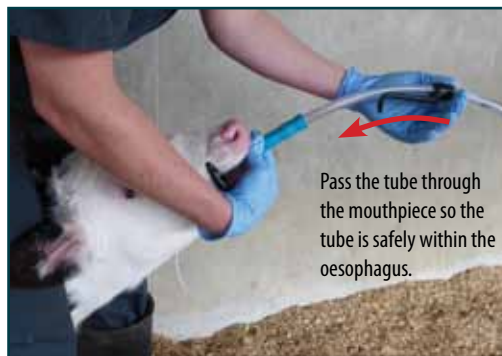

Pass the tube through the mouthpiece so the tube is safely within the oesophagus.




Secure tube with finger hook

When using the strap:  
Before passing the tube, place the strap behind the ears + pull the tab to fit comfortably.



Pass the tube into the oesophagus and maintain tube position.

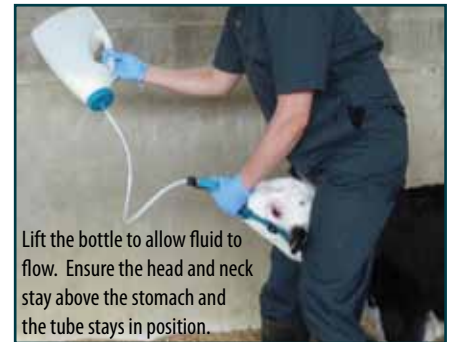

Lift the bottle to allow fluid to flow. Ensure the head and neck stay above the stomach and the tube stays in position.

- Flexible, transparent tube.
- Easy swallow safety tip.
- Easy to clean.
- Fast flow, even with thick colostrum.
- Calf size markers.

### IMPORTANT

There should never be resistance when passing the tube.

If there is resistance:

**STOP**, withdraw the tube, relax the calf's head, aim slightly left and then gently pass again.



### SPARES:



ANT10C

ANT20C




# Stress + Struggle Free Calf Tube Feeding

from



Kinder  
Easier  
Safer  
Faster

## TRUSTI TUBER 2 IN 1 STARTER KIT (ANT01A)

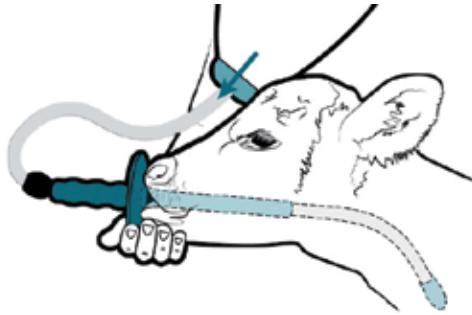

With 88% reduction in calf stress and discomfort (compared with rigid feeders), the Trusti and Flexi Tubers allow you to tube feed your calves with ease and peace of mind.

The combination of mouthpiece, flexible tube and safety tip makes tube feeding easier, safer and more comfortable.

As proven in farm trials, a smoother process makes a faster process and a better experience for farmer and calf.



### STOPPER POSITION AND CALF SIZE MARKERS:

- Consider the calf size markers a **MINIMUM** distance to pass the tube through.
- The stopper should be positioned to the **RIGHT HAND SIDE** of the appropriate mark.



Do not underestimate calf size as it is better to pass the tube farther than not far enough. If feeding a group of calves we recommend you set the stopper position to the largest calf in the group.



### IMPORTANT

Never let fluid flow unless the tube is safely in the oesophagus.

### SPARES:



Tel: +44 (0) 1948 667676  
 Fax: +44 (0) 1948 666505  
 Email: sales@dairyspares.com  
 Website: www.dairyspares.com

