

**Dairy
Spares**

**HAND/MANUAL
MILK PUMP
FOR SHEEP
AND GOATS**



KUG01A



HAND MILK PUMP FOR SHEEP AND GOATS

The Hand Milk Pump for Sheep and Goats is ideal for:

- mastitis cases
- when a ewe or goat doesn't go in for milking
- when the animal withholds milk from the new born

The kit comprises of:

- 1 x Pump
- 2 x Bottle (500ml)
- 2 x Silicone liners (1 x sheep, 1 x goat)
- 1 x Teat, rubber and fits bottle
- 1 x Sample of Milkers Gloves



Installation and use:

Place silicone liner in white plastic base.

Screw bottle to the base.

Gently secure the assembly to the pump system.

Place the liner over the teat and pump using the hand to milk.

Once milking finished, remove the bottle.

Screw cap on bottle to store in fridge or use teat provided to feed kid.



Cleaning:

Rinse with hot water after each use.

Clean with alkaline, as used with milking machines.